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**INSTRUCTION FOR USE**  
Please read carefully before using

**SPECIFICATIONS**

Range: 28.0 °C to 42.9 °C; Below 28.0 °C: Display «Lo»; Above 42.9 °C: Display «Hi»

Intended use: Measurement of body temperature

Resolution: 0.1 °C

Accuracy: ± 0.1 °C between 35.5 °C and 42.9 °C; ± 0.2 °C above 35.5 °C at an ambient temperature of 18 °C to 28 °C

Display: Liquid crystal display 3 1/2 digits

Battery (included): Micro Alkaline 192 or LR41 or AG3; 1.5 V

IP classification: IP 22 (effective against solid foreign objects with a finger; protected against dripping water)

Dimensions: Ca. 134.5 mm x 18.7 mm x 10.7 mm

Weight: Ca. 14 g including battery

Beep: To control the power on and when the measurement is completed.

Memory: Built-in memory enabling display of the previously measured value

Working Conditions: Temperature: 10 °C to 45 °C; Humidity: 15 % to 95 % R.H.

⚠ It is only possible to measure temperatures that are above the ambient temperature.

Storage Conditions: Temperature: -25 °C to +55 °C; Humidity: 15 % to 95 % R.H.

Guarantee of Quality: Conformity DIN EN 12470-3, Medical Thermometers part 3. Compact electrical thermometers with maximum device. It is certified in accordance with Council Directive 93/42/EWG and EN ISO 13485 and is entitled to affix the CE-mark **CE** 0197 (Notified Body: TÜV Rheinland LGA Products GmbH).

Electromagnetic compatibility (EMC): The present medical product is a device with extremely high degree of noise immunity. Therefore, the enclosure with the EMC tables was not printed out.

**HOW TO OPERATE**

- Press down the control button to start the thermometer. Then, a full display **[000]** ensures that the LCD window is in good function.
- The next appearance **[Lo]** signals normal operation after internal safe testing. Otherwise **[Err]** alerts its malfunction.
- Then, the ambient temperature or **[Hi]** displays on window with a full display. The thermometer is now ready for a temperature taking. If there is no temperature taking in the coming 3 minutes, the thermometer will be automatically shut-off.
- Clean or disinfect the probe before every measurement. Put the probe head under the tongue, under the arm or in the rectum; according to the methods provided (see Methods of Use).
- As soon as the thermometer does not register any further relevant increase in temperature, "C" stops flashing and the maximum value measured up to that time is shown, accompanied by a series of audible signals.
- The thermometer continues measuring the temperature for a further minute after the audible signals begin, and in this time there may possibly be a further slight increase in the temperature shown. After that, the maximum temperature measured is recorded and stored; it remains visible on the LCD display for 8 minutes until the thermometer is automatically switched off.
- When the measured temperature is above or below measuring range, the LCD window displays **[Lo]** or **[Hi]**.

**HOW TO RECALL THE LAST TEMPERATURE**

If you hold the button for more than 3 seconds when switching on, the last temperature will be recalled on the LCD window.

At the same time, a small „M“ for memory appears. After 3 more seconds, this value and the „M“ disappear and the thermometer is in normal measurement mode.

**METHODS OF USE**

⚠ It is only possible to measure temperatures that are above the ambient temperature.

**Axillary Measurement** (approximately 60 - 120 seconds)  
Dry the armpit with a towel. Place the probe head under the armpit. Close the arm firmly. In general, the axillary temperature is around 0.5 °C to 1.0 °C lower than oral temperature. Note that in order to achieve better axillary temperature measurement result, a minimum measuring time of 3 to 5 minutes is recommended regardless of the beep sound.

**Oral Measurement** (approximately 45 - 90 seconds)  
Place the probe head under the tongue. Assume a fixed and good contact. Close the mouth and breathe normally.  
A normal temperature by either method is around 36.0 °C and 37.5 °C.

**Rectal Measurement** (approximately 30 - 60 seconds)  
Lubricate the probe head with Vaseline for an easier insertion. Then, softly enforce the probe head into the rectum approximately 2 to 3 cm in depth. In general, the rectal temperature is around 0.5 °C to 1.0 °C higher than oral temperature.

If you compare temperatures between two thermometers, it is possible to have two different readings even if both thermometers are within their accuracy tolerances. Differences in how you take each reading and the thermometer's tolerance range can account for nearly 0.1 °C variability.

**RECOMMENDATION: Determine your normal temperature and watch for changes.**

**HOW TO REPLACE THE BATTERY**

When the battery flag **[Batt]** appears and blinks, it means time to replace the battery. In this case,

- Turn the thermometer over, with the label side facing to the user. Pull out the cover as the following illustration.
- Remove the used battery by any pointed object but not sharp metal.
- Withdraw the battery.
- Insert a new 1.5 V button battery with the cathode side (-) up.
- Close the cover securely.

**Warning:** Do not dispose the battery in normal domestic waste! Out of reach of young children. Do not dispose of the battery in a fire. It may explode!

**CLEANING AND DISINFECTION**

The thermometer is waterproof. Please use one of the following disinfectants for disinfecting, following the manufacturer's instructions in each case.

Disinfectant	Dilution with water	Method
Ethanol	70 - 80 %	Wipe the probe with a cloth moistened with the disinfectant.
Isopropyl Alcohol	70 - 90 %	Wipe the probe with a cloth moistened with the disinfectant
Glutaraldehyde	0.5 - 2 %	Immerse the probe for 30 minutes
Alkylpolyaminoethylglycine	0.05 - 0.2 %	Immerse the probe for 30 minutes
Benzalkonium Chloride	0.1 %	Immerse the probe for 30 minutes
Benzathonium Chloride	0.1 %	Immerse the probe for 30 minutes
Glucuronate Chlorhexidine	0.1 - 0.5 %	Immerse the probe for 30 minutes
Sodium Hypochlorite	0.02 - 0.05 %	Immerse the probe for 30 minutes

**Note:** Any other cleaning solution, dilution or cleaning method can cause a malfunction of the thermometer! The thermometer must never be boiled!

**PRECAUTIONS**

- Use of this instrument is not intended as a substitute for consultation with your physician. Prolonged fever requires medical attention, especially for your children. Please contact your physician.
- Child's temperature shall be taken under adult's surveillance.
- Keep out of the reach of unperervised children. Choking resulting from a child swallowing a small part that has become detached from the thermometer.
- Do not boil, bite, bend, drop or disassemble this thermometer.
- Do not attempt to disassemble the unit except to replace the battery.
- Avoid taking temperature in 30 minutes after physical exercises, bathing or dining.
- Do not store, use and transport the unit under direct sunlight, at a high temperature, or in high humidity or dust. Performance may be degraded.
- The possibility of allergic reactions and / or skin irritation cannot be excluded completely.
- If the thermometer has been stored at below-freezing temperatures, allow the thermometer to warm naturally to room temperature before using.

**DISPOSAL**

Observe the applicable regulations when disposing of the thermometer and batteries. This thermometer must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please inform the batteries before disposing of the thermometer. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop.

**WARRANTY**

This thermometer is guaranteed for two years from the date of purchase against manufacturer's defect under normal use. If your unit does not function properly due to defective parts or assembly we will repair it free of charge. All parts are covered by this warranty excluding the battery. The warranty does not cover damage to your unit due to improper handling. We recommend that you have checked the thermometer every 2 years by an authorized laboratory if used professionally (required by law in Germany).

**SYMBOL INDEX**

Follow instructions for use	Type BF applied part	Manufacturer
Keep dry	Caution! Read the instruction manual.	Batch code (mm/yyyy, month/year)
The device must not be disposed of with household waste	Store between 15 % and 95 % R.H.	Store between -25 °C and +55 °C

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